USER MANUAL FOR HYGENETEC FAECES BAGS.

HOW TO USE

The Hygenetec faeces bags are designed for collecting human waste. Each Hygenetec bag contains a super absorbent gel pad which will absorb up to 650ml of fluid in a short period of time. Hygenetec super absorbent gel pads reduce splashing and decrease odour whilst minimising the risk of cross-contamination. After use simply tie the bag tightly and safely dispose in a waste bin.

WARNING: Keep away from children. Plastic bags can cause suffocation. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wash hands thoroughly after using a Hygenetec bag.

STEP-BY-STEP USER MANUAL FOR HYGENETEC FAECES BAGS

STEP 1: Open the box and unroll the Hygenetec faeces bag. Tear it off along the perforated edge.

STEP 2: Open the bag, place it over the bedpan and fold it over the edge. Turn the bedpan over and tighten the cords. Place into the commode chair ready for use.

STEP 3: Make sure that the absorbent pad is on the bottom of the bedpan.

STEP 4: Wait for approximately 30 to 60 seconds until the absorbing pad (\pm 650ml) is changed into a gel substance.

STEP 5: Close the bag by pulling the zip fastener. Tie the bag on the top to make sure it's fully closed.

STEP 6: Dispose of the bag with the regular waste or according to the protocol of your institution.

This product is latex free and the gel pads are bio-degradable and PH neutral. Contents: **20 pieces** of faeces bags + absorption pads.

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