USER MANUAL FOR HYGENETEC URINAL BAGS.

HOW TO USE

The Hygenetec urinal bags are designed for collecting human waste. Each Hy- genetec bag contains a super absorbent gel pad which will absorb up to 650ml of fluid in a short period of time. Hygenetec super absorbent gel pads reduce splashing and decrease odour whilst minimising the risk of cross-contamination. After use simply tie the bag tightly and safely dispose in a waste bin.

WARNING: Keep away from children. Plastic bags can cause suffocation. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wash hands thoroughly after using a Hygenetec bag.

USED IN
HEALTHCARE

VERY EASY
TO USE

SUPER
ABSORBENT

LIMITED RISK
OF INFECTION

KEEP AWAY
FROM CHILDREN

STEP-BY-STEP USER MANUAL FOR HYGENETEC URINAL BAGS

STEP 1: Open the box and unroll the Hygenetec urine bag. Tear it off along the perforated edge.

STEP 2: Open the bag, put an absorption pad in the bag and hold the bag open with both hands.

STEP 3: Urinate in the bag.

STEP 4: Wait for approximately 30 to 60 seconds until the absorbing pad (\pm 650ml) is changed into a gel substance.

STEP 5: Close the bag by pulling the cords. Tie the bag on the top to make sure it's fully closed.

STEP 6: Dispose of the bag with the regular waste or according to the protocol of your institution.

This product is latex free and the gel pads are bio-degradable and PH neutral. Contents: **20 pieces** of urinal bags + absorption pads.

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